

Verifying & Setting the Time and Date

Why: Sets the time and date for your specific area.

Where: SET UP I on the SET TIME screen.

How:

- From the TIME OF DAY screen, press SEL to SET UP I then press ACT. You will see the SET TIME screen. Press ACT, "PROG HOURS" will appear.
- Use the arrows to select the hour, then press ACT. The screen will say "PROG MINUTES".
- Use the arrows to select the minutes and press ACT. The screen will say "PROG YEAR".
- Use the arrows to select the year and press ACT. The screen will say "PROG MONTH".
- Use the arrows to select the month and press ACT. The screen will say "PROG DAY".
- Use the arrows to select the day and press ACT.

3:06 PM
SET TIME

3:06 PM
PROG
HOURS

4:06 PM
PROG
MINUTES

4:07 PM 99
PROG
YEAR

4:07 PM 12
PROG
MONTH

4:07 PM 04
PROG
DAY

- The time and date that you just set will be displayed.

4:07 PM
04DEC99

 **NEED TO KNOW**

The 508 does not have an On/Off button. As soon as you install batteries, the pump will be On and the current time will be shown on the window display.

WHY IS THE TIME AND DATE NECESSARY?

- ✗ The time shown on your clock is used to time the insulin delivery of your basal rate(s) and to monitor other pump operations.
- ✗ The date is used to sort the long term data stored in the pump's memory.
- ✗ This stored information can be downloaded to a computer for you or your health care professional to use in assisting you in your glucose control goals.