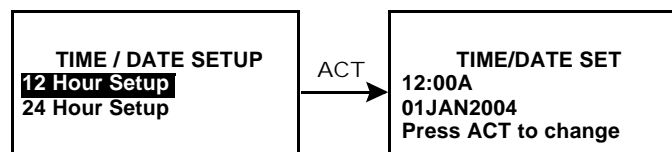




Setting the time and date

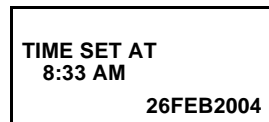
Setting the correct time and date in your pump is necessary for accurate basal insulin delivery and allows you to keep an accurate record of your insulin delivery and other pump functions. You can select a 12-hour or 24-hour clock. You must reset the time and date if you receive a CHECK SETTINGS alarm or you clear your settings (Clear Settings function).

- Go to the TIME/DATE SETUP screen.
Main > Utilities > Time/Date
- Select **12-Hour Setup** or **24-Hour Setup** and press ACT.
- Press ACT again to change the settings.
- Change each of the settings as follows:



Hour	Minutes	Year	Month	Day
SET HOUR 12:00A	SET MINUTES 9:00A	SET YEAR 2004	SET MONTH 01	SET DAY 01
Change the hour. Press ACT. For 12-hour setups, press  /  until the correct A (am) or P (pm) appears.	Change the minutes. Press ACT.	Change the year. Press ACT.	Change the month. Press ACT.	Change the day. Press ACT.

- 5 The TIME SET AT screen will show the settings that you programmed. Press ACT and exit the menus.



Your time/date settings are complete.

Bolus

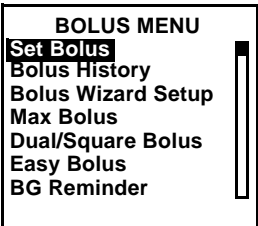

There are three bolus types: Normal, Square Wave, and Dual Wave. This section gives instructions for a Normal bolus using the express bolus button and navigating through the menus. (For information about Square Wave and Dual Wave boluses, refer to chapter 6, “Optimizing pump therapy.”)

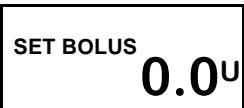
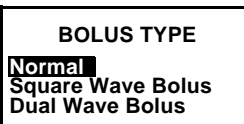
The Normal bolus delivers an immediate food or correction bolus. It can be delivered at any time except during another Normal bolus. During a Normal bolus, most pump features are disabled until after all the bolus has been delivered. The suspend function and the STATUS screen, however, are always available.

Normal bolus

Normal bolus can be used to cover the carbohydrate in a meal or snack and/or to correct a blood glucose that is higher than the target that was chosen for you.

The following instructions are for a Normal bolus without the Bolus Wizard feature.

from the menu, or	using the EXPRESS BOLUS button
<p>1 Go to the BOLUS MENU. Main > Bolus</p> <p>Select Set Bolus and press ACT. Go to step 2.</p> 	<p>Press  on your pump. Go to step 2.</p>

<p>2 If the SET BOLUS screen appears: (Dual/Square option is off) Go to step 3.</p> 	<p>If the BOLUS TYPE screen appears: (Dual/Square Wave is on) Select Normal Bolus and press ACT. Go to step 3.</p> 
---	---

- 3 Enter your bolus amount and press ACT.

